

# Rystora®

IN HARMONY WITH YOUR SKIN



## *Silicone Scar Sheet* *(Non-Sterile)*




REDUCE THE SIZE AND COLORATION OF  
HYPERTROPHIC AND KELOIDAL SCARS

---

Rystora® Silicone Scar Sheet is a safe, effective and reusable treatment for scars and keloids at the joint and other flat skin. It enhances skin rejuvenation following recent surgery, scalds, burns or minor skin injuries, as well as effectively reduces the size, hardness, itchiness and discoloration of scars.



MEDICAL GRADE SILICONE

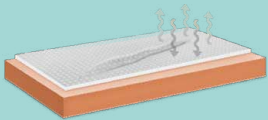


BREATHABLE AND ANTI-IRRITATION



VIRTUALLY PAINLESS TO REMOVE



FLEXIBLE AND EASY TO APPLY TO JOINT

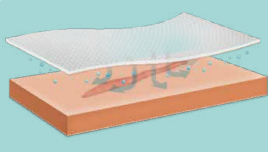

KEEP SKIN MOISTURIZED



WASHABLE AND REUSABLE

## DIRECTIONS FOR USE

- Ensure that the skin is clean, dry, and free of ointment or cream before applying.
- Rystora® Scar Sheet can be cut to fit smaller scars or combined to cover larger scars.
- For best results, Rystora® Scar Sheet should be applied consecutively for a minimum of 12 hours per day and for at least 8 weeks.
- Rystora® Scar Sheet is reusable and can be used for at least 7 days if washed properly and dried daily.

## SPECIFICATION

REF. No.	Description	
SA050701	Small: 2"x3" (5x7.5cm)	1 pc/case
SA050712		12 pcs/case
SA101801	Regular: 4"x7" (10x18cm)	1 pc/case
SA101812		12 pcs/case
SA042501	Strips: 1.6"x10" (4x25cm)	1 pc/case
SA042512		12 pcs/case

# Rystora®

**Fortune Medical Instrument Corp.**

6 F., No. 29, Sec. 2, Jhongjheng E. Rd., Danshuei Dist,  
New Taipei City 251, Taiwan

Manufactured by: No. 256, Changchun 2nd, Rd.,  
Jhongli Dist, Taoyuan City 320, Taiwan

T. +886 2 2624-2233 [www.rystora.com](http://www.rystora.com)

Rystora® is Registered Trademark of Fortune Medical.

EDM SHEET Eng. V20220531